Your Needs, our Innovation

Collection and transport system for microbiological samples

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Range of δ swab[®] system Gama del sistema δ swab[®]

Looking at the new needs of the market and endeavouring to make microbiology departments and analysis laboratories' jobs easier, Deltalab has developed a new range of swabs for collecting and transporting microbiological samples in liquid medium as well as a new line of enrichment media.

Advantages:

- Compatible with the new inoculation automation and sample streaking equipment.
- Compatible with molecular diagnostic techniques.
- Facilitates the collection, transport and subsequent treatment of microbiological samples.
- Greater reliability for sample recovery: high capacity for the absorption and elution of the sample from the new flocked swab, δ swab[®]
- Greater feasibility of the sample as it is totally suspended in the medium.
- Allows different homogeneous inocula to be obtained from the same sample.
- It adapts to any working protocol.
- Minimises any possible cross-contamination: less handling and maximum watertightness.
- More comfort for the user: conical base which facilitates stirring and greater stability of the tube with skirt base.
- More comfort for the patient, due to the softer swab covering.
- Standardises the receipt of samples at the microbiology labs.
- Facilitates manual streaking.
- Allows direct extension on the glass slide as the medium does not contain agar.
- Ensures proper sample transport and storage both at room temperatures (20°C - 25°C) and at refrigeration temperatures (4°C - 8°C).

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
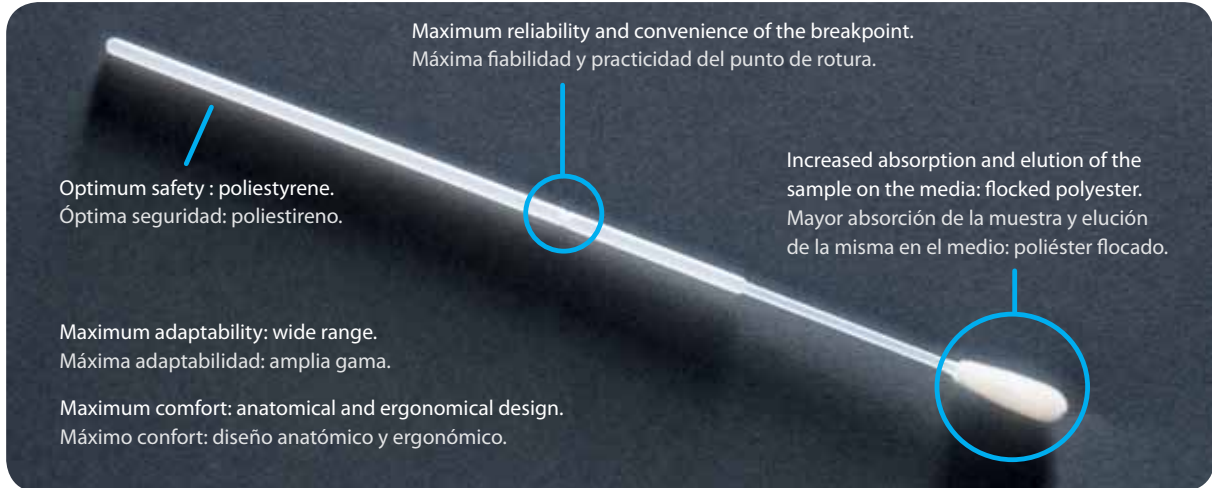
Deltalab offers a wide range of flocked swabs, with or without breaking point, that perform the most efficient absorption and elution for the collection of microbiological samples. These swabs are presented in peel-pack without transport medium. For codes and features information, please ask our sales department.

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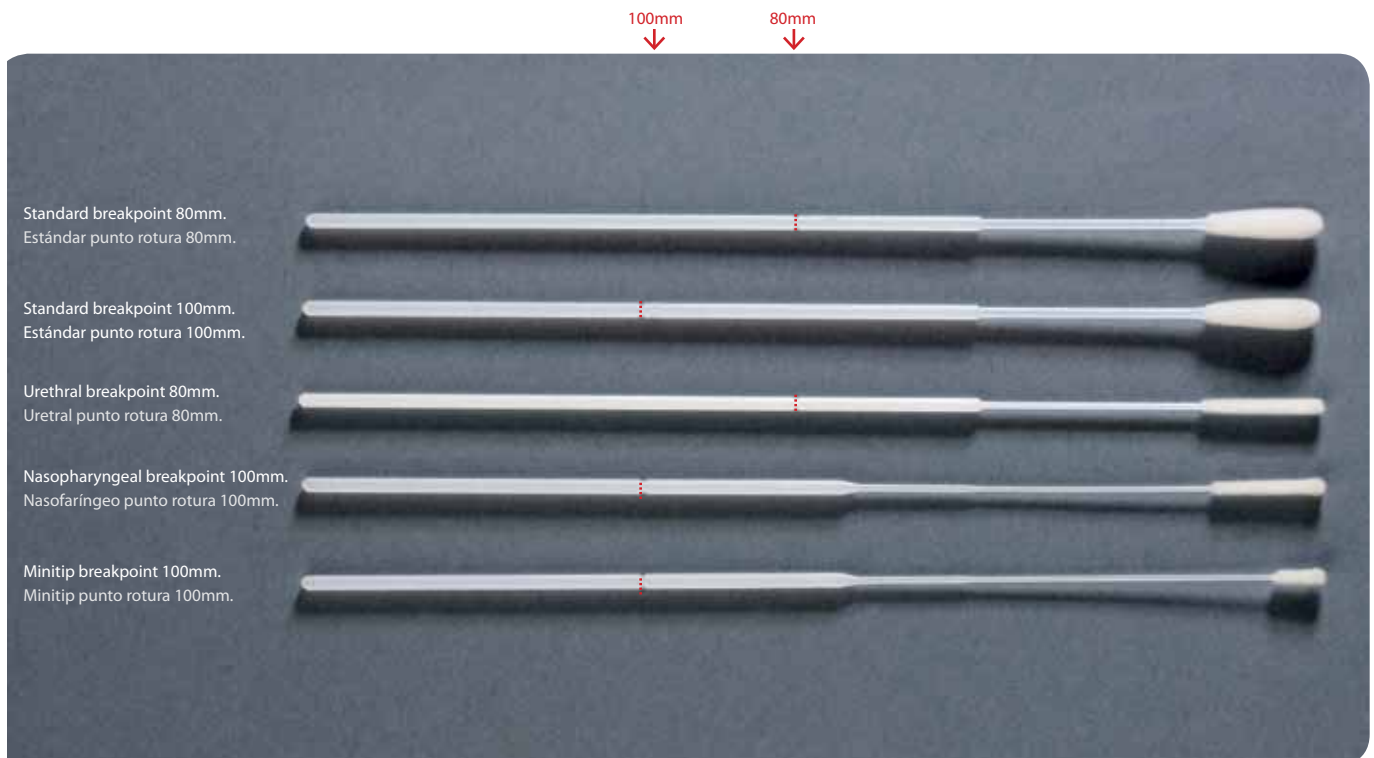Particularly suited for microbiological sample collection and transport. It maintains the viability of Aerobes, Facultative Anaerobes and Anaerobes Bacteria for a minimum of 48 hours both at room temperature (20-25°C) and at refrigeration temperature (4-8°C) and 24 hours with Fastidious Bacteria, following CLSI, M40-A2* standard methodology.

Designed both for traditional and automated sample streaking. Compatible with the molecular diagnostic techniques and for direct extension on glass slides.

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* M40-A2: CLSI Quality Control of Microbiology Transport System: Approved Standard - Second Edition

photo foto	code código	ml ml	description descripción	label etiqueta	breakpoint p. rotura	case qty. cant. caja	case wt. peso caja	case vol. vol. caja	package present.
1	304281	1 ml	With standard flocked swab Con escobillón flocado estándar	[Blue]	80mm	6 x 100	5.7	0.056	peel-pack
2	304282	1 ml	With urethral flocked swab Con escobillón flocado uretral	[Yellow]	80mm	6 x 100	5.6	0.056	peel-pack
3	304285	1 ml	With nasopharyngeal flocked swab Con escobillón flocado nasofaríngeo	[Light Blue]	100mm	6 x 100	5.6	0.056	peel-pack
4	304286	1 ml	With minitip flocked swab Con escobillón flocado minitip	[Red]	100mm	6 x 100	5.5	0.056	peel-pack

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Specially suited for the collection and transport of faecal samples. Maintains the viability of the fecal pathogens for a minimum of 48 hours without overgrowth, following CLSI, M40-A2* standard methodology. Designed for carrying out sample collection directly from the rectum or from faeces.

Designed for traditional and automated sample streaking. Compatible with the molecular diagnostic techniques by PCR and for direct extension on glass slides.

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Particularly suited for collecting and transporting microbiological samples which contain Virus, *Chlamydia*, *Ureaplasma* and/or *Mycoplasma*. Includes antibiotics which inhibit the growth of Bacteria and Fungi; this ensures reliability in terms of sample recovery. Maintains the infectability of viruses for a minimum of 96 hours both at room temperature (20-25°C) and at refrigeration temperature (4-8°C), following CLSI, M40-A2* standard methodology. Each tube contains glass beads to facilitate cell lysis, sample homogeneity and maximize the elution of the sample from the swab. Designed for cell cultivation and compatible with molecular diagnostic techniques by quantitative PCR.

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Particularly suited for collecting and transporting microbiological samples which contain viruses. By including antibiotics which inhibit the growth of Bacteria and Fungi, it ensures the trustworthiness of the sample. It maintains the infectability of viruses for up to 96 hours, both at room temperature (20-25°C) and at refrigeration temperature (4-8°C), following CLSI, M40-A2* standard methodology.

Designed for cell cultivation and to be compatible with the molecular diagnostic techniques by quantitative PCR.

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Deltalab has developed its own range of enrichment media. These media contains the nutrients required to promote the growth of a wide variety of microorganisms.

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SELENITE SELENITO

IVD

Inhibits the proliferation of bacterial microbiote. It is particularly suited for isolating *Salmonella* from faecal samples, food and/or water. It is used as an enrichment media for clinical and industry samples.

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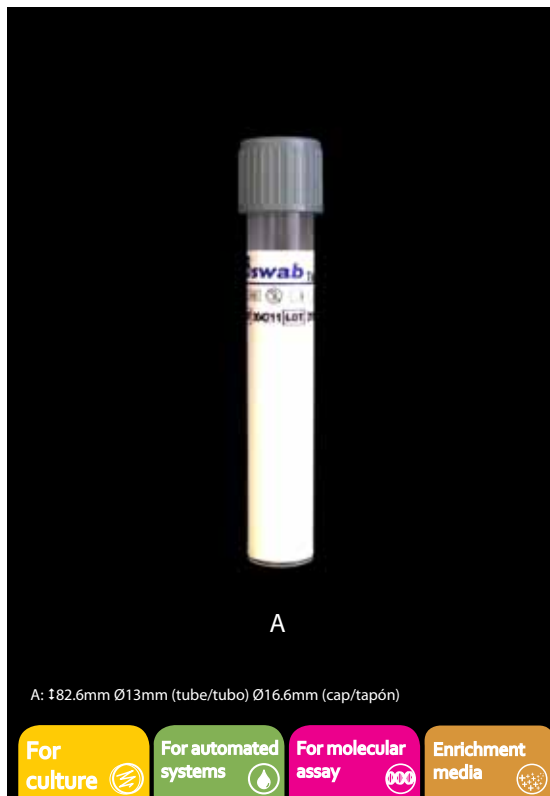
THIOGLYCOLAT TIOGLICOLATO

IVD

Widely used as enrichment media for isolation and cultivation of Aerobic and Anaerobic Bacteria and Fastidious Bacteria. It is used as an enrichment media for clinical samples.

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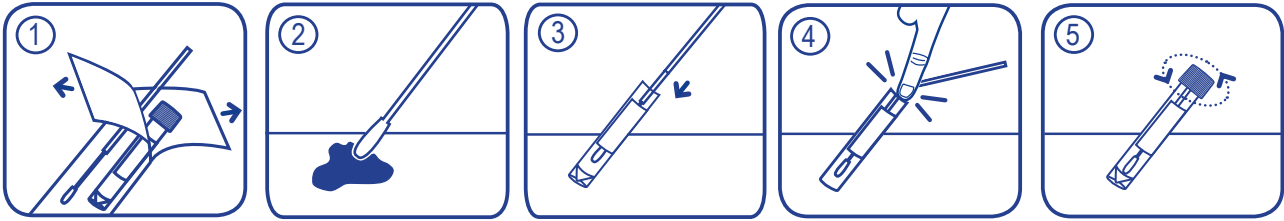
Packing Presentación

- ① The AMIES, CARY BLAIR and VIRUS transport media, are supplied in peel-pack plastic + plastic with a small slit at the top to allow easy opening. Inside there is a swab and a tube with medium, both are sterile by radiation.

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- ② ViCUM® transport media is supplied in a peel bag made of medical paper + plastic. Inside there is a swab inside a plastic flow-pack, sterile by radiation, and a tube with medium, sterile by filtration.

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- ③ Enrichment media are supplied in a box with internal rack to place 60 tubes.

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Innovative solutions in

Microbiology
Molecular Biology
Haematology
Histology
Sample handling
Safety
General Labware

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